


## *LoopMaster-Sochman Snare*

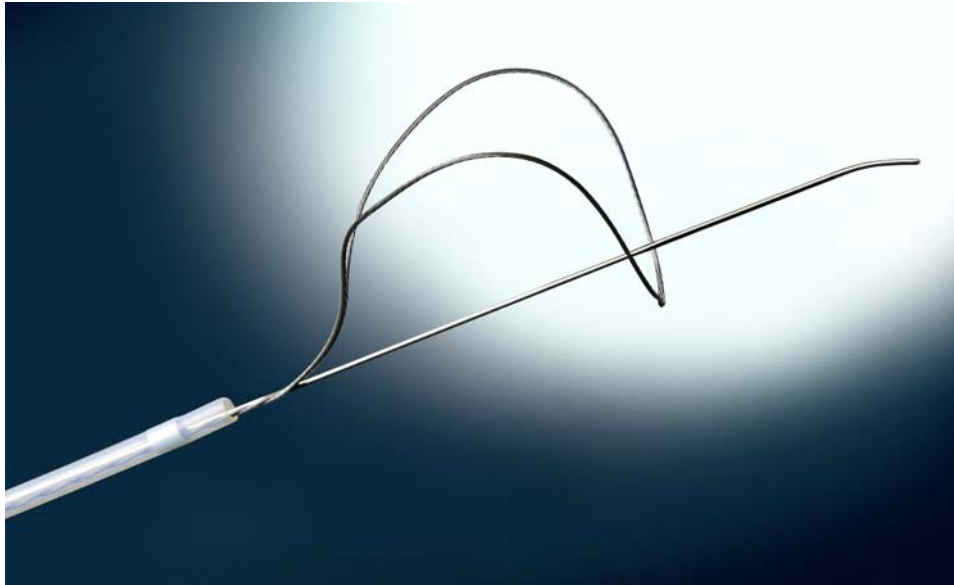

The LoopMaster-Sochman snare from Nuros has been specifically designed to facilitate the retrieval of foreign bodies when an open end is not accessible. A simple but highly effective solution.

- Atraumatic distal guide wire tip
- Nitinol snare loop
- Integral wire
- Radiopaque platinum inner marker
- Highly flexible shaft
- Supportive sleeve
- Extra 'BIGG' torque device to assist manipulation

Procedure:

1. Using the loop controller, advance the snare loop and manipulate it into place around the foreign body
2. Using the guide wire controller advance the guide wire to locate between the object and the device loop extremity
3. Pull back the loop controller to close the loop, trapping the foreign body between the snare and the guide wire.
4. The foreign body may then be retrieved.

Ordering Information:

Product Code	Loop Diameter (mm)	Length (cm) Loop/GW	Introducer Sheath (cm)	Introducer sheath (F)
ASL-25	25	130/150	110	8